



# MEDISEPT

Perfect disinfection



## MEDISEPT Dry Wipes

### Dry wipes for soaking with disinfecting/cleaning agent

MEDISEPT Dry Wipes are designed for soaking with a ready-to-use product or utility solution of concentrate with water. Formulated for cleaning and disinfecting various surfaces of non-invasive medical devices. The wipes may be used in healthcare facilities, surgeries, nursing homes, and other areas, according to the intended use of the soaking agent.

#### The features and benefits of using wipes for soaking:

- The porous structure of the wipes with grammage of 45 g/m<sup>2</sup> perfectly absorbs the agent guarantees its ideal distribution on the cleaned surface.
- The thick unwoven cloth does not scratch the surface.
- Non-dusting and lint-free white wipes.
- Very enduring and resistant to detergents and solvents.
- Recommended soaking: 1 litre per bucket allows for effective disinfection of 2 m<sup>2</sup> surface with one wipe.
- The wipe size is 16 x 30 cm.
- The optimal number of wipes (100 pcs) allows their effective use for up to 30 days (solution-storage time).
- Perforated wipes are easy to separate and pull out of the bucket.
- Tightly packed in foil protects the wipes against moisture and contamination.
- The durable and solid reusable bucket is easy to disassemble and clean.
- The handy bucket makes it easy to carry while working.
- The tight bucket lid protects the soaked wipes from drying out and spilling the solution during moving.



Dry Wipes  
100 pcs



For small and  
large surfaces



Durable and convenient bucket


# MEDISEPT Dry Wipes

## Direction for use


Prepare the soaking liquid according to the recommendations on the product's label. To soak the wipes in the liquid, follow the instructions as shown in the pictograms below. Wipe the surface with a damp wipe and leave it to dry for the time recommended by the soaking-liquid's manufacturer. Clean the bucket and lid before re-use with the soaking solution. Using the liquid-soaked wipes does not require protective gloves.



1 Open the bucket lid.  
Unpack the roll from the foil (discard the foil).

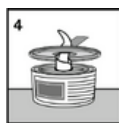

4 Close the bucket and the opening in the lid.  
Leave for 30 minutes for all wipes to soak fully.

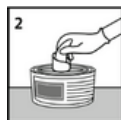

2 Remove the first wipe from the roll inside.  
Pull through the opening in the lid.



5 Fill in the sticker on the bucket.  
When using the ancillary insert,  
place the sticker on the bucket.



3 Fill the bucket with the chosen solution  
of the cleaning/disinfecting product to the  
desired concentration or with the ready-to-  
use product. Amount of soaling liquid: 1 litre.

## Storage:

Spray the surface with the product from the distance of approximately 30 cm, ensuring it is entirely covered, and leave it for the formulation to take effect. The surface dries without any smudges left. Do not use it on delicate alcohol-sensitive surfaces (e.g. acrylic glass). Professional use only.

<b>Composition of nonwovens:</b>	polypropylene (50%), cellulose (50%)
<b>Wipe grammage:</b>	45 g/m <sup>2</sup>
<b>Wipe size:</b>	16 x 30 cm
<b>Bucket material:</b>	HDPE with high-resistance to chemicals

## Packaging units:

bucket with wipes for soaking insert, 100 pcs  
supplementary insert for the bucket, 100 pcs

## Products recommended for use with MEDISEPT Dry Wipes:



Velox® Spray

Velox® Top AF

Velox® Foam Extra

Quatrodes® Extra

Quatrodes® Forte

Medical device class II a within the meaning of the Act on Medical Devices.  
The bucket is not a medical device.

