



MEDISEPT

Perfect disinfection



MEDISEPT Dry Wipes

Dry wipes for soaking with disinfecting/cleaning agent

MEDISEPT Dry Wipes are designed for soaking with a ready-to-use product or utility solution of concentrate with water. Formulated for cleaning and disinfecting various surfaces of non-invasive medical devices. The wipes may be used in healthcare facilities, surgeries, nursing homes, and other areas, according to the intended use of the soaking agent.

The features and benefits of using wipes for soaking:

- The porous structure of the wipes, with a grammage of 55 g/m² perfectly absorbs the agent, and guarantees its ideal distribution on the cleaned surface.
- The thick unwoven cloth does not scratch the surface.
- Non-dusting and lint-free white wipes.
- Very enduring and resistant to detergents and solvents.
- The wipe size is 16 x 30 cm, with a recommended soaking of 1 litre per bucket, allows the effective cleaning of 2m² of the surface.
- The optimal number of wipes (100 pcs) allows their effective use for up to 30 days (solution-storage time).
- Perforated wipes are easy to separate and pull out of the bucket.
- Tightly packed in foil protects the wipes against moisture and contamination.
- The durable and solid reusable bucket is easy to disassemble and clean.
- The handy bucket makes it easy to carry while working.
- The tight bucket lid protects the soaked wipes from drying out and spilling the solution during moving.



Dry wipes
100 pcs



For all surfaces



Durable and convenient bucket


MEDISEPT Dry Wipes

Direction of use:


Prepare the soaking liquid according to the recommendations on the product's label. To soak the wipes in the liquid, follow the instructions as shown in the pictograms below. Wipe the surface with a damp wipe and leave it to dry for the time recommended by the soaking-liquid's manufacturer. Clean the bucket and lid before re-use with the soaking solution. Using the liquid-soaked wipes does not require protective gloves.




1 Open the bucket lid.
Unpack the roll from the foil (discard the foil).

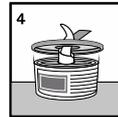

4 Close the bucket and the opening in the lid.
Leave for 30 minutes for all wipes to soak fully.

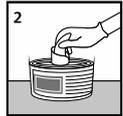

2 Remove the first wipe from the roll inside.
Pull through the opening in the lid.

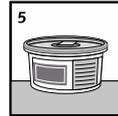

5 Fill in the sticker on the bucket.
When using the ancillary insert,
place the sticker on the bucket.



3 Fill the bucket with the chosen solution
of the cleaning/disinfecting product to the
desired concentration or with the ready-to-use
product. Amount of soaling liquid: 1 litre.

Storage:

Keep the wipes in the original packaging in the tightly closed bucket at 5°C-25°C. Protect from moisture and sunlight. After soaking, store for up to 30 days.

Composition of nonwovens:	polypropylene (50%), cellulose (50%)
Wipe grammage:	55 g/m ²
Wipe size:	16 x 30 cm
Bucket material:	HDPE with high-resistance to chemicals

Packaging units:

- bucket with wipes for soaking insert, 100 pcs, EAN 5907626638035
- supplementary insert for the bucket, 100 pcs, EAN 5907626638059

Products recommended for use with MEDISEPT Dry Wipes:

Velox Spray
Velox Top AF
Velox Foam Extra
Quatroles Extra
Quatroles Forte
Quatroles Strong
Quatroles One



The compatibility of products with the wipes has been confirmed by microbiological-efficiency test using the four-field test according to the EN 16615 standard

Medical device class II within the meaning of the Act on Medical Devices.
The bucket is not a medical device.



 Producer
MEDISEPT Sp. z o.o.
Konopnica 159 c
21-030 Motycz

phone: + 48 81 535 22 22
export@medisept.pl
www.medisept.pl

Revision: 08.2020