



MEDISEPT

Perfect disinfection



Quatrodes® Forte

Concentrate for surfaces cleaning and disinfection

Quatrodes® Forte is a liquid concentrate for simultaneous cleaning and disinfection of surfaces in the medical area (units, operating chairs and beds, operating tables, medical equipment, floors, walls), in gastronomy, nursing homes, sanatoriums, gyms, fitness and SPA centres, hairdressing, massage and beauty salons and solariums (for disinfecting sunbeds), tattoos parlours, kindergartens and schools. The product is suitable for surfaces having contact with food.

Product features:

- Concentration from 0.5% in 15 min.
- Active against Bacteria, Fungi, Mycobacteria, enveloped viruses, Adeno, Polio.
- Recommended for units, beds and treatment chairs, operating tables, medical equipment, floors.
- Does not require rinsing with water.
- For disinfecting surfaces having contact with food.
- 1 L of concentrate = 200 L of working solution.
- Registration: medical device and biocidal product.

User benefits:

- Guaranteed effectiveness and safety of use.
- High reduction of the infection risk.
- Quick effect of clean and disinfected surface.
- Can be used on many sensitive materials.
- Universal, dual use for medical and non-medical surfaces.
- Economical in usage.
- Fast and convenient in preparing the solution.



Concentrate



Active against
B, F, V, Tbc,
Adeno, Polio



Medical
surfaces



Concentration
from 0.5%
in 15 min.

Quatrodes® Forte

Application:

Quatrodes® Forte is a liquid concentrate for simultaneous cleaning and disinfection of surfaces in the medical area (units, operating chairs and beds, operating tables, medical equipment, floors, walls), in gastronomy nursing homes, sanatoriums, gyms, fitness and SPA centres, hairdressing, massage and beauty salons and solariums (for disinfecting sunbeds), tattoos parlours, kindergartens and schools. The product is suitable for surfaces having contact with food. Product has a bactericidal, mycobactericidal, yeast- and virucidal effect. The substance can be used on all floor and above floor surfaces (doors, tabletops, cabinets, tables) and for cleaning and disinfecting hairdressing accessories.

Effectiveness	EN14885	Time	
		15 min.	30 min.
Bacteria (including MRSA)	EN 13727	0,5%*, 1%	
	EN 14561	1%	0,5%
	EN 13697	1%*	
Fungi (C. albicans)	EN 13624	0,5%	
	EN 14562	1%	0,5%
	EN 13697	1%*	
Mycobacteria (M. terrae, M. avium)	EN 14348, EN 14563	4%	1%
Enveloped viruses (Vaccinia, BVDV, HBV, HIV, HCV, Herpes simplex, Influenza A, Ebola, SARS-Cov-2)	EN 14476	0,5%	
Adeno virus	EN 14476	1%	
Polio virus	EN 14476	4%	

* clean conditions

Directions for use:

Prepare a working solution by diluting the concentrate with water in a desired proportion. The choice of concentration depends on the desired spectrum and duration of operation. In order to obtain a concentration of 1% dilute 10 ml of the concentrate in 990 ml of water. Disinfected surfaces do not require rinsing with water. Do not combine with other detergents nor disinfectants. For professional use only.

Active substances in 100 g of the product:

3.76 g of N- (3-aminopropyl) -N-dodecylpropane-1,3-diamine (CAS: 2372-82-9),
3.39 g of Poly (oxy-1,2-ethanedyl), alpha. [2- (didecylmetyloamino) ethyl] - omega.-hydroxy-, propanoate (salt) (CAS: 94667-33-1),
0.75 g 2-aminoethanol (CAS: 141-43-5)

Packaging units:

1 L bottle with doser,
5 L canister



Medical device class II within the meaning of the Medical Devices Act. Intended for disinfection of non-invasive medical devices.

Permission number for biocidal products in Poland: 6066/15

The safety data sheet and the component data sheet are available at www.medisep.pl

CE 2274

 Manufacturer:
MEDISEPT Sp. z o.o.
Konopnica 159 c
21-030 Motycz

phone: + 48 81 535 22 22
export@medisept.pl
www.medisep.pl

Revision: 08.2020