



# MEDISEPT

Perfect disinfection



## Quatrodes® Unit NF

### Concentrate for cleaning and disinfection of suction devices

The concentrate is designed for simultaneous cleaning and disinfection of suction systems, amalgam separators and dental unit bowls.

#### Product features:

- Concentration from 1% in 15 min.
- Active against Bacteria, Fungi, Mycobacteria, enveloped viruses.
- Recommended for suction systems, dental unit bowls, amalgam separators.
- Does not foam.
- For disinfection of a closed circuit in hydromassage tubs.
- It prevents biofilm from forming.

#### User benefits:

- Can be used on many sensitive materials.
- Guaranteed effectiveness and safety of use.
- High reduction of the infection risk.
- Effectively removes dried organic residues.
- Fast and convenient in usage.



Concentrate



Active against  
B, F, V, Tbc



Dental unit



Concentration  
from 1%  
in 15 min.

# Quatrodes® Unit NF

## Application:

The concentrate is designed for simultaneous cleaning and disinfection of suction systems, amalgam separators and dental unit bowls. The product does not foam. Used daily, it prevents biofilm from forming. It can also be used to disinfect closed circuit in hydromassage tubs that are medical devices. For professional use only.

Effectiveness	EN14885	Time		
		15 min.	30 min.	60 min.
Bacteria (including MRSA)	EN 13727	1%		
Fungi ( <i>C. albicans</i> )	EN 13624	1%*		
	EN 13624	2%	1%	
Tuberculosis ( <i>M.terrae</i> )	EN 14348			4%
Enveloped viruses (HBV, HIV, HCV, Ebola, Vacinnia, Influenza, Herpes simplex, SARS-Cov-2)	EN 14476	2%		

\* clean conditions

## Directions for use:

The concentrate should be diluted with water. In order to obtain 2% concentration, dilute 20 ml of the product in 980 ml of water.

- **In suction system:** Introduce the solution to the suction system, depending on the degree of soiling, once or twice a day. Suck the whole product, turn the device off and leave it for at least 15 minutes. After disinfection, rinse the suction system with water. Rinsing can be carried out after the recommended disinfection time or the next day, immediately after turning the unit on. Do not combine with other detergents nor disinfectants.
- **In hydromassage tub:** Before the first use, rinse the device with water to remove any possible residues of other product. Fill the tub with clean, cold water to the level above of the nozzles. Add 200 ml of the product for every 10 L of water. Turn the hydromassage system on for 15 minutes. After use, empty the tub and rinse with water. Do not combine with other agents. Do not use the product during the patient's healing bath.

## Ingredients: 100 g contains:

4.4 g dioctyl-dimethylammonium chloride (CAS: 5538-94-3),  
0.6 g dodecyl-dimethyl-benzylammonium chloride (CAS: 85409-22-9)

## Packaging units:


1 L bottle with doser,  
5 L canister



Medical device class II within the meaning of the Medical Devices Act. Intended for disinfection of non-invasive medical devices.

The safety data sheet and the component data sheet are available at [www.medisept.pl](http://www.medisept.pl)

CE 2274

 Producer:  
**MEDISEPT Sp. z o.o.**  
Konopnica 159 c  
21-030 Motycz

phone: + 48 81 535 22 22  
[export@medisept.pl](mailto:export@medisept.pl)  
[www.medisept.pl](http://www.medisept.pl)

Revision: 08.2020