



MEDISEPT

Perfect disinfection



Velox® Duo Wipes

Alcoholic wipes for cleaning and rapid disinfection of small surfaces and medical equipment

Wipes for cleaning and rapid disinfection of small surfaces in medical field, nursing homes, sanatoriums, fitness and SPA centres, hairdressing, massage and beauty salons, kindergartens and school. The product is suitable for surfaces having contact with food.

Product features:

- Active in 30 secs against Bacteria, Fungi, Tuberculosis, enveloped viruses, Adeno and Rota viruses.
- Recommended for medical equipment, rehabilitation equipment, dental handpieces, patient beds, cabinets, tabletops.
- For disinfection of medical devices surfaces having contact with food.
- Non-woven wipes 19x15 cm, 50 g/m².
- Dermatologically tested

User benefits:

- Guaranteed effectiveness confirmed by tests.
- High reduction of infection risk.
- Rapid effect of clean and disinfected surface.
- Refreshing effect.
- Economical in usage.



Wipes



Active against
B, F, Tbc, V
in 30 secs



Non-invasive
medical devices



Tea tonic scent

Velox® Duo Wipes

Application:

Wipes for cleaning and rapid disinfection of small surfaces in medical field, nursing homes, sanatoriums, fitness and SPA centres, hairdressing, massage and beauty salons, kindergartens and school. The product is suitable for surfaces having contact with food. Recommended for small surfaces of medical equipment: operating beds and chairs, beds, medical and surgical equipment and disinfection of dental tips before sterilisation process, prosthetic spoons and silicon moulds. The wipes disinfect surfaces killing in 30 seconds bacteria, fungi, mycobacteria, enveloped viruses (such as Vaccinia, BVDV, SARS-CoV-2, HIV, HBV, HCV), Rota and Noro viruses.

Effectiveness	EN 14885	Conditions	
		clean	dirty
Bacteria (including MRSA)	EN 13727	30 sec.	30 sec.
	EN 16615	30 sec.	30 sec.
Fungi (C. albicans)	EN 13624	30 sec.	30 sec.
	EN 16615	30 sec.	30 sec.
Mycobacteria tuberculosis (M. terrae)	EN 14348	30 sec.	30 sec.
Enveloped viruses (HIV, HBV, HCV, Vaccinia, SARS-CoV-2, Herpes simplex, Ebola)	EN 14476	30 sec.	60 sec.
BVDV virus	EN 14476	30 sec.	30 sec.
Rota virus	EN 14476	30 sec.	60 sec.
Adeno virus	EN 14476	60 sec.	30 sec.

Directions for use:

Take a wipe out of the package and wipe the surface. Wait 30 seconds. In dirty conditions operating time against enveloped viruses and Rota virus is extended up to 60 seconds. In case disinfecting surfaces having contact with food, rinse them with potable water. Other surfaces do not require rinsing with water. Close the package tightly. After opening the package, use the wipes within 21 days. Do not use if the wipe is dry. Do not use on surfaces sensitive to alcohol (e.g. acrylic glass). For professional use only.

Ingredients: 50 pcs contain

250 g of soaking agent.

100 g of soaking agent contains:

ethanol, propan-2-ol

Packaging units:

50 wipes in a flow pack with self-adhesive closing



Permission number for biocidal products in Poland: 7905/19

Medical device class II a within the meaning of the Medical Devices Act.
Intended for disinfecting non-invasive medical devices.

The safety data sheet and the component data sheet are available at www.medisept.pl

CE 2274

 Responsible entity:
MEDISEPT Sp. z o.o.
Konopnica 159 c
21-030 Motycz, Poland

phone: + 48 81 535 22 22
export@medisept.pl
www.medisept.pl

Revision: 10.2020