



MEDISEPT

Perfect disinfection



Velox® Wipes NA

Alcohol-free wipes for cleaning and disinfecting sensitive surfaces of non-invasive medical equipment

Velox® Wipes NA are recommended for disinfecting surfaces of non-invasive medical equipment made of plastic, acrylic glass, stainless steel, metal, aluminum, rubber and porcelain.

Product features:

- Active in 1 min. against Bacteria, Fungi, enveloped viruses and in 5 min. against Tuberculosis.
- Active in 5 min. against Mycobacteria.
- For surfaces sensitive to alcohol.
- Destined for disinfection of various medical equipment surfaces made of glass, porcelain, steel, rubber, plastic and acrylic glass, operating chairs, rehabilitation equipment, incubators.
- Recommended for ultrasound lamps and heads.
- Non-woven wipes 13x20 cm, 23 g / m².

User benefits:

- Guaranteed effectiveness and safety of use.
- High reduction of the infection risk.
- Rapid effect of clean and disinfected surface.
- Can be used for many sensitive materials.
- Refreshing effect.
- Economical in usage.



Wipes



Active in 1 min.
against B, F, V



Non-invasive
medical devices



Alcohol-free
product

Velox® Wipes NA

Application:

Velox® Wipes NA are recommended for disinfecting surfaces of non-invasive medical equipment made of plastic, acrylic glass, stainless steel, metal, aluminium, rubber and porcelain. Suitable for disinfection of medical appliances, rehabilitation equipment, treatment chairs, incubators, the heads of ultrasound machines and lamps in intensive care units and operating theatres. Free from alcohol, aldehydes and phosphates, the wipes do not discolour the surfaces. The product has a pleasant scent. The product is suitable for medical equipment surface having contact with food. Tested and recommended by Famed - a leading European producer of top quality hospital equipment.

Effectiveness	EN14885	Conditions	
		clean	dirty
Bacteria (including MRSA)	EN 16615	60 sec.	3 min.
	EN 13727	60 sec.	60 sec.
Fungi (C. albicans)	EN 13624	60 sec.	60 sec.
	EN 16615	60 sec.	60 sec.
Mycobacteria tuberculosis (M. terrae)	EN 14348	5 min.	10 min.
Enveloped viruses (HIV, HBV, HCV, Vaccina, Influenza, Vaccinia, BVDV, Herpes simplex, SARS-Cov-2)	EN 14476	60 sec.	60 sec.

Directions for use:

Place the refill in the box. Take a tissue out of the box and wipe a surface to be cleaned and disinfected. Close the lid. After opening the package, use the tissues within 30 days. Do not use if a tissue is dry. The surface does not need to be rinsed with water. Before applying on particularly sensitive surfaces, test the wipes on a non-visible area. For professional use only.

Ingredients:

100 pieces of tissues contain: 173 g of saturated formulation

100 g contains:

0.15 g N- (3-aminopropyl) -N-dodecylpropane-1,3-diamine, 0.14 g Poli (oxy-1,2-ethanedyl), alpha .- [2- (didecylmethylamino) ethyl] - omega. hydroxy, propanoate (salt)

Packaging units:


100 pieces box,
100 pieces refill



Medical device class II within the meaning of the Medical Devices Act. Dedicated for disinfecting non-invasive medical devices.

The safety data sheet and the component data sheet are available at www.medisept.pl

CE 2274

 **Manufacturer:**
MEDISEPT Sp. z o.o.
Konopnica 159 c
21-030 Motycz

phone: + 48 81 535 22 22
export@medisept.pl
www.medisept.pl

Revision: 08.2020